



## F&A Desobtur

Solvent for the root canal desobturation

*F&A Desobtur – solvent for the root canal desobturation.*

### INDICATIONS FOR USE:

- For softening and desobturation of the root canals sealed with the phenol, formalin, and formaldehyde resins.
- Disobturation (unsealing) of root canals after sealing with phenol-formaldehyde-based resins.

### PRODUCT BENEFITS:

- High efficiency of unsealing.
- Contains flavoring additives.
- Presence of surfactant facilitates the penetration of the solvent into the recesses and cavities

### METHOD OF USING:

- Desobturation of the canal is performed by combined chemical action of F&A Desobtur and mechanical action of endodontic instruments
- Process the tooth chamber and entrance of the canal.
- Place a drop of F&A Desobtur at the mouth of the canal.
- Delete the contents of the canal using the appropriate file, wetting the instrument with F&A Desobtur.
- Repeat the procedure step by step until the apex is reached.

### RECOMMENDATIONS AND PRECAUTIONS:

- The canal which is unsealed with F&A Desobtur must be thoroughly dried before subsequent filling.
- Avoid contact with eyes and oral soft tissue. If accidental contact with eye occurs rinse immediately with large amount of water.

**PACKING:** F&A Desobtur in 9 ml bottle.

